



ZRHY Medical

Huaian Hongyu Medical Instruments Co.,ltd
ADD:46 Dukangqiao Road, ,Huai 'An City, Jiangsu Province, China
Tel:+86-+86-13511523530
E-mail:info@hymedicals.com;
Official Site: <http://www.hymedicals.com>

Hongyu Medical Instruments Operating Instruction

Instruction Details:

● All Hongyu Medical instruments are non-sterile products and it must be sterilized before use. The correct sterilization methods as follows:

① High-pressure steam sterilization : pressure steam sterilizer is always used for sterilization, rise the pressure to 102.9kPa (1.05kg /m²), the temperature 121℃~126℃ ,stay for 20-30 mins to achieve sterilization purpose.

②Cauterization sterilization: The method is usually taken when the urgent situation happens. Place the instruments in the basin, pour a small amount of 95% alcohol and ignite for 3 to 5 minutes to sterilize.

*Note: This method has a damaging effect on the device. Oxacyclopropane can also be used for sterilization when necessary.

● After use, the instruments should be cleaned and disinfected, and the boiling method can be used (After cleaning and disinfection,the instruments could be put into boiling water 100℃ for 10 minutes)

The reusable instruments could be be cleaned with ultrasound.

*Note:

1).Hongyu Medical instruments should be apart from other instruments and the head needs to be protected during cleaning.

2). When the abrasion,rusting,breakage happens unluckily,the instruments must be deserted.

- Medical Use

*Storage Conditions:

The instruments must be stored at the draughty and cleaning place without corrosion air. And they must be kept dry always,prevent being wet.

*Operating Instruction:

All the instruments of Hongyu Medical must be washed with cleaning water and disinfected with ethyl alcohol before being put into the sealing operating box. Before the operation,all the instruments must be sterilized again.